

CryoPen® o



● Ø 1-3mm | ○ Ø 2-6mm | ● Ø 4-8mm
Power: ■ | Power: ■■ | Power: ■■■

With better control of procedures the CryoPen O sets new standards for superb outcomes.

The CryoPen O dispenses a fine jet of nitrous oxide under high pressure which allows you to work with millimeter precision.

It offers the ideal solution for accurate, quick, effective stress free treatment of all skin imperfections.

Instrument

This instrument is well prepared for all treatment options.

Potential Users

Dermatologists, GPs, Aesthetics Doctors, ENT-specialists, Pediatricians, Veterinarians, Urologists and Gynecologists.

Cartridges

8g N₂O cartridges: 1 box containing 24 cartridges
1 carton containing 12 boxes

16g N₂O cartridges: 1 box containing 6 cartridges
1 carton containing 12 boxes

Once punctured the pressure will remain for weeks. Shelf life 24 months.
Storage at room temperature.

CryoPen O set with 3 applicators

Red/blue dot applicator: Ø 1-3mm
flow time 110-140sec (8g) / 250-280sec (16g)

Red/white dot applicator: Ø 2-6mm
flow time 80-100sec (8g) / 180-210sec (16g)

Red/green dot applicator: Ø 4-8mm
flow time 65-80sec (8g) / 145-165sec (16g)

What kind of lesions?

All benign lesions: Warts, actinic & seborrheic keratosis, hemangioma, molluscum contagiosum, condyloma, plantar warts, and more...

About the procedure

No special preparation is required. Treatment is applied for 2 to 25 seconds on each spot. In larger areas circular or crossing movement is required.